SEALBOND ULTIMA

INSTRUCTIONS FOR USE OF THE SEALBOND ULTIMA SYSTEM After ETCHING

INSTRUCTIONS FOR USE

DESCRIPTION :

Light-cured bonding system, with ideal thickness ($< 10 \mu\text{m}$) and highly light sensitive. Does not interfere with dual cure cement. Ultima can be used with one shot* technique with Light Post® and DT-Light Post® and also in two shots** technique with any kind of fiber post.

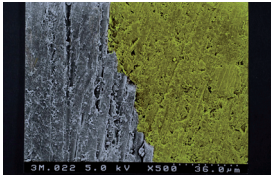
CHARACTERISTICS :

SEALBOND II ETCHING 32%

Optimum concentration liquid etchant for dentine preparation

SEALBOND ULTIMA

. Single component, 5th generation bonding system.
. Bonding system to seal and cement fiber post and crown reconstitution

Interface post/cement

* one shot means cure the cement and primer through the post at the same time

** two shot means light cure the primer inside the canal first, then place the post and light cure the exposed cement around the post.

Priming

Dispense 2 drops Sealbond Ultima.
Apply 2 consecutive coats to canal and coronal dentin without waiting between coats, using a Composit-brush especially designed for that purpose.
Remove excess of primer with paper point.

Apply one coat onto the post
Evaporate solvent and water with an air syringe (air dry) for 10 seconds.
Light-cure 10s.
Light cure the root canal and coronal dentine for 20 seconds (400mW/cm²).
In case of Light post® or DT-Light post®, this can be done in one shot when light-curing the cement through the post, during cementation.

STORAGE :

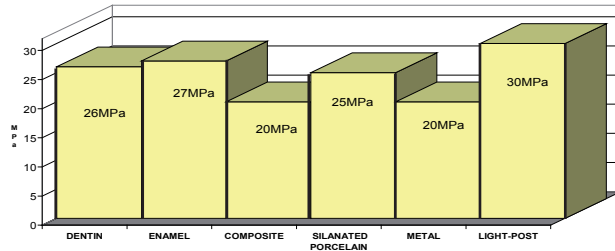
Store all components in a cool place preferably in a refrigerator.
Do not use products after the expiry date, displayed on the packaging.

RECOMMENDATIONS:

IMMEDIATELY recap bottles after using to avoid solvent evaporation.
Use an isolating technique such as rubber dam etc...
Avoid contact between products and skin, mucous membrane and eyes. Protective gloves, safety glasses and mask are recommended.
If contact occurs with eye, rinse immediately with water and seek medical advice if necessary. Wash with water and soap if contact occurs with skin.

The material has been developed for use in dentistry only and must be carried out following the instructions and for the indications. In no event will the manufacturer be responsible for damages of any nature whatsoever resulting from the use of or reliance upon information. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk in connection.

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